

VOCO Profluorid® Varnish

YOUR CHOICE

Indications

Treatment of hypersensitive teeth

Sealing of the dentinal tubules on sensitive root surfaces or following cavity preparation

Advantages

- Easy to apply on wet surfaces
- Pleasant melon flavour
- Aesthetic, white-transparent tooth colour
- Fast desensitisation and fluoride release (5% NaF ≙ 22,600 ppm fluoride)

Advantages of the *SingleDose*

- Product and applicator in hygienic disposable packing
- No additional devices required
- No spilling in any position
- Available in four flavours: melon, mint, cherry and caramel

NOW AVAILABLE
IN THREE ADDITIONAL
FLAVOURS






Presentation

- REF 1267 Tube 10 ml melon
- REF 1268 50 *SingleDose* 0.25 ml melon, Easy Brush
- REF 2225 48 *SingleDose* 0.40 ml mixed, Easy Brush
- REF 1269 50 *SingleDose* 0.40 ml melon, Easy Brush
- REF 2226 50 *SingleDose* 0.40 ml caramel, Easy Brush
- REF 2227 50 *SingleDose* 0.40 ml cherry, Easy Brush
- REF 2228 50 *SingleDose* 0.40 ml mint, Easy Brush

- REF 1271 200 *SingleDose* 0.40 ml melon, Easy Brush
- REF 2229 200 *SingleDose* 0.40 ml caramel, Easy Brush
- REF 2230 200 *SingleDose* 0.40 ml cherry, Easy Brush
- REF 2231 200 *SingleDose* 0.40 ml mint, Easy Brush

VOCO GmbH
P.O. Box 767
27457 Cuxhaven
Germany
Tel.: +49 (0) 4721-719-0
Fax: +49 (0) 4721-719-140
info@voco.de
www.voco.com

Available from:

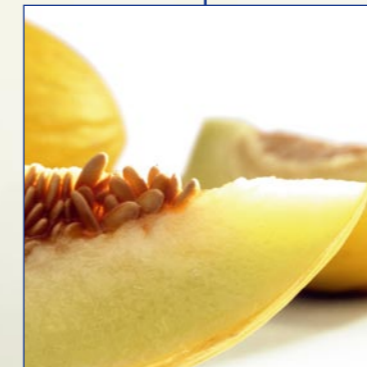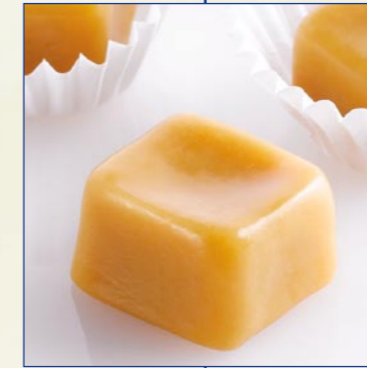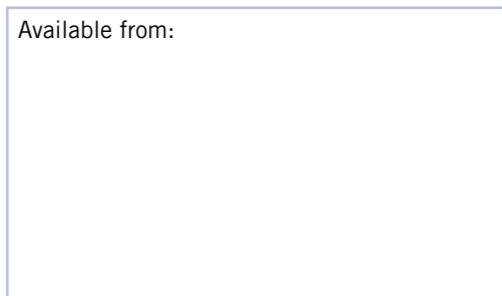

VOCO Profluorid® Varnish

DESENSITISING VARNISH WITH FLUORIDES

VOCO Profluorid® Varnish

HIGHLY EFFECTIVE

Hypersensitivity


Sensitive teeth are a widespread problem for a large number of people. Usually, the condition causes only transient pain which patients do not always report to their dentists. Hypersensitivity is mainly caused by exposed dentinal tubules. Therefore, the aim of treatment is to seal these tubules.

Indications


VOCO Profluorid® Varnish is a fluoride-containing varnish for treating hypersensitivity. Dentinal tubules exposed on the necks of teeth or after cavity preparation are effectively sealed.

Components and effect of VOCO Profluorid® Varnish

VOCO Profluorid® Varnish is a colophony-based varnish containing 5% sodium fluoride (22,600 ppm fluoride). The fluoride ion, together with the calcium ions accumulated in the tubules, causes a precipitation of calcium fluoride, which effectively seals the tubules. According to a user survey, over 85% of patients were completely free of pain after only one treatment with VOCO Profluorid® Varnish.

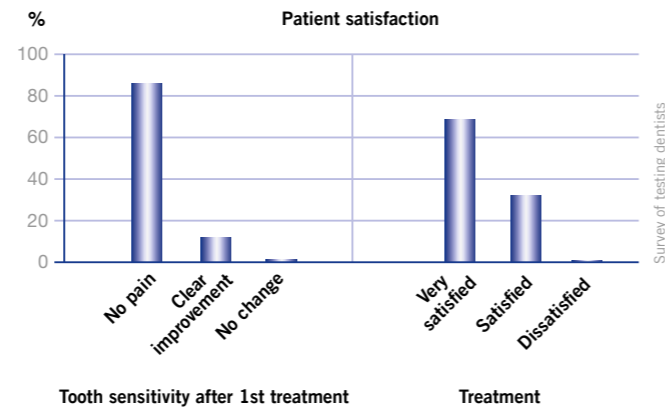

As well as rapidly sealing the dentinal tubules, VOCO Profluorid® Varnish also causes calcium fluoride deposits to form on the tooth surface. These deposits protect the tooth from acid attacks, promote remineralisation and contribute to the long-term formation of fluoroapatite. Xylitol, which has a proven cariostatic effect in addition to its taste-enhancing properties, is also added to the varnish.

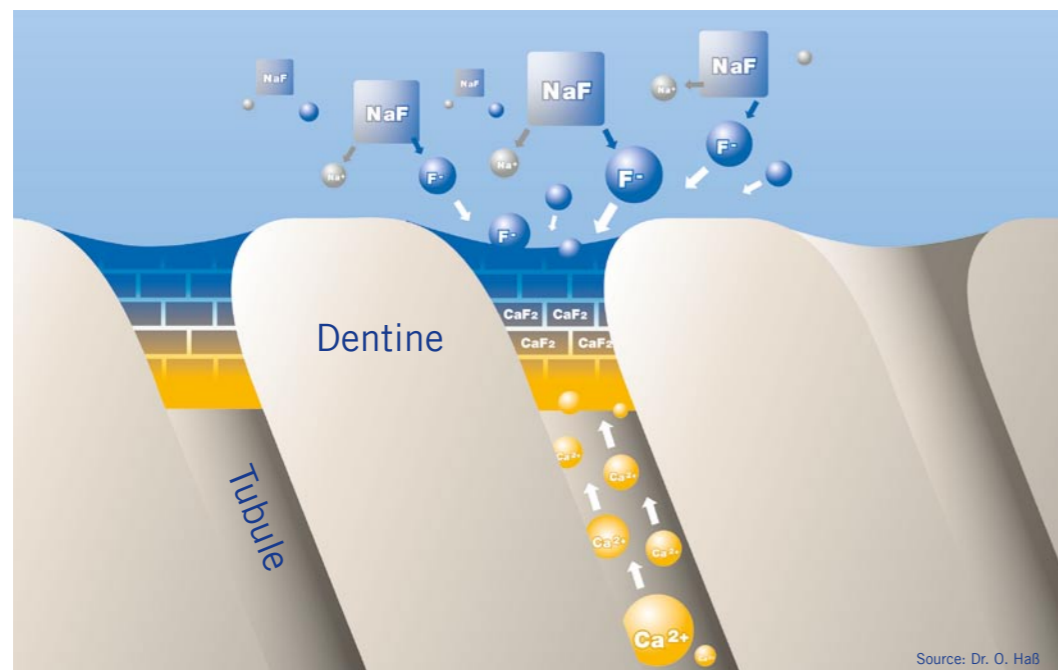

Source: Dr. O. Haß

VOCO Profluorid® Varnish


SIMPLE AND AESTHETIC

Easy to apply

Thanks to its colophony matrix, VOCO Profluorid® Varnish adheres well even to wet surfaces. It is not necessary to dry the tooth surfaces. This improves cooperation of patients with low levels of compliance (e.g. children).

Aesthetics and taste

VOCO Profluorid® Varnish is a tooth-shaded varnish which does not impair aesthetics. VOCO Profluorid® Varnish is available in four pleasant flavours: melon and cherry - aimed especially at children and younger patients, mint - a classic, particularly for adults, and caramel - to provide a slightly more exotic option. These four refined flavours will enable you to provide an even better service by tailoring it to the individual patient.

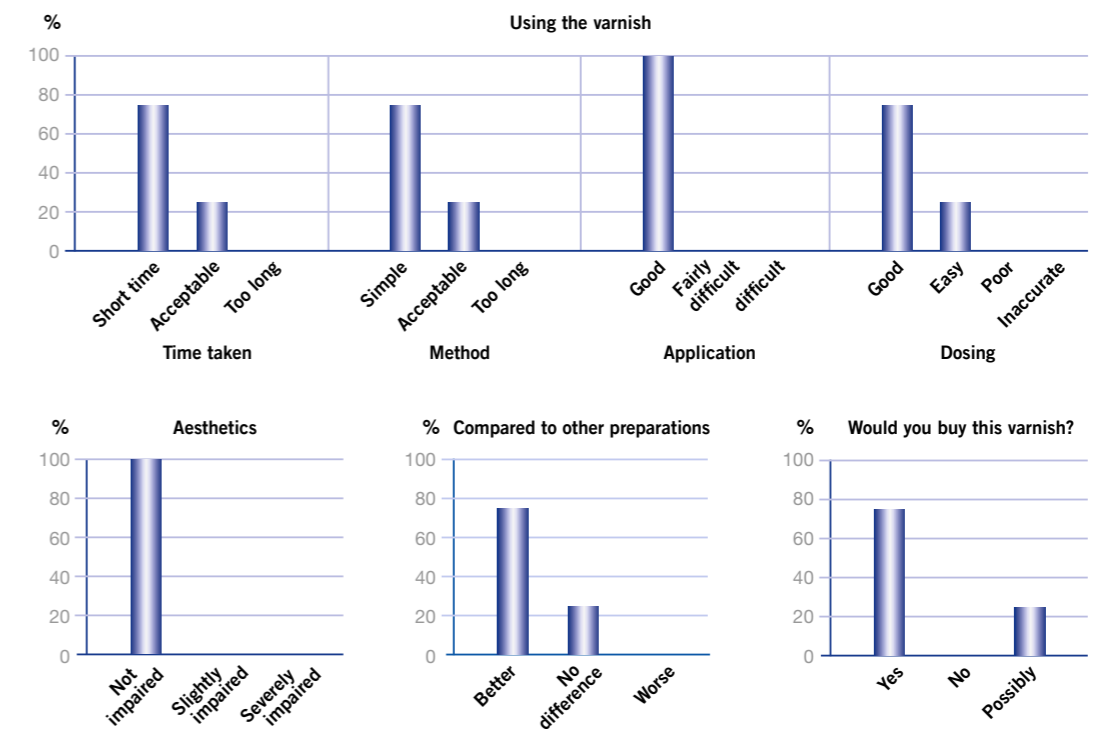

Opinion of dentists

Dentists using the product all confirmed its good handling and aesthetic properties. A vast majority also reported positive differences between Profluorid Varnish and competing products. VOCO Profluorid® Varnish was equally well accepted by the patients. After the treatment, nearly all of the patients reported that they were either symptom-free or had a significantly reduced level of sensitivity.

VOCO Profluorid® Varnish: much more than just good taste!

The SingleDose advantages

VOCO Profluorid® Varnish *SingleDose* comes in a blister pack containing a brush and the varnish. This enables the product to be applied under perfectly hygienic conditions.

Simple and aesthetic desensitising with VOCO Profluorid® Varnish:

